



Pressure Bar

STOP UNCONTROLLABLE BLEEDING



4" width - NSN #6510-01-460-0849
6" width - NSN #6510-01-492-2275

Choose the bandage you can rely on for severe traumas:

Knife Wounds | Gunshot Wounds | Lacerations | Fallen Elderly | Colleague Injuries | Car Accidents



1 Place pad on wound.



2 Insert cloth leader into pressure bar.



3 Pull and wrap in opposite direction.



4 Continue to tighten/wrap bandage.



5 Clip closure bar onto bandage.



5310 Elm Street, Houston, Texas 77081 | Tel 713-723-6000 | Fax 713-723-6221
Toll Free 1-888-737-7978 | www.ps-med.com | info@ps-med.com