

EMERGENCY BANDAGE

C²

Compression Bandage
Civilian Bandage

Choose the bandage you can rely on for severe traumas:

- Knife Wounds
- Gunshot Wounds
- Lacerations
- Fallen Elderly
- Colleague Injuries
- Car Accidents

STOP UNCONTROLLABLE BLEEDING



Product Numbers:

- 4" C2 White Bandage: #FCP-03
- 6" C2 White Bandage: #FCP-06
- 4" C2 White Bandage w/ Sliding Pad: #FCP-11
- 6" White Bandage w/ Sliding Pad: #FCP-12
- 12x12" White Abdominal Bandage: #FCP-10



Pressure Bar

The Emergency Bandage has been clinically proven to produce up to 60% more pressure than bandages without the patented Pressure Applicator.



1 Place pad on wound.



2 Insert cloth leader into pressure applicator.



3 Pull and wrap in opposite direction.



4 Continue to tighten / wrap bandage.



5 Clip closure bar onto bandage.